

Urine Analysis

Medi-Test Combi 10[®] SGL

Optimized
Urine Test Strip



Desiccant enclosed in cap

⊕ Convenient and safe

Best available vitamin C protection

⊕ Safe, early diagnosis of serious diseases

Premium color charts perfectly match reaction colors

⊕ Clear read-off, reliable results

⊕ Quality made in Germany

Since its foundation in 1911, Macherey-Nagel have developed Filter papers into top quality test papers for laboratory use and offer Word's largest selection of different test papers. In late seventies MN become forth manufacturer to produce high quality Urine test strips and today customer in 70 countries trust outstanding quality of this test strips.

Fugen
B I O M E D






Medi-Test Combi 10® SGL

EN ISO 13485 AND EN ISO 9001 certified

Glucose Protein/Albumin Ketones Leukozytes Blood
Nitrite Density pH value Bilirubin Urobilinogen


Globally renowned Urine test strip producer for more than 30 years

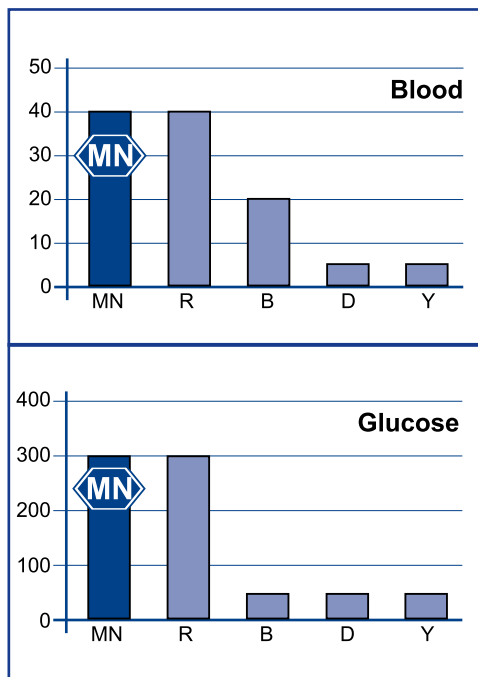

Best available vitamin C protection


Comparative data clearly show: Combi 10® SGL urine test strips have the best protection of blood and glucose tests against vitamin C. With other brands, vitamin C may prevent the oxidation reaction essential for these test. This may cause false negative results for blood and glucose. Using Medi-Test Combi 10® SGL, **blood and glucose** in urine **will be safely detected**. Patients can receive treatment at the earliest stage thus increasing chances for success.

u Concentration of vitamin C in the sample with no effect on the results for blood and glucose in urine (mg/dL)

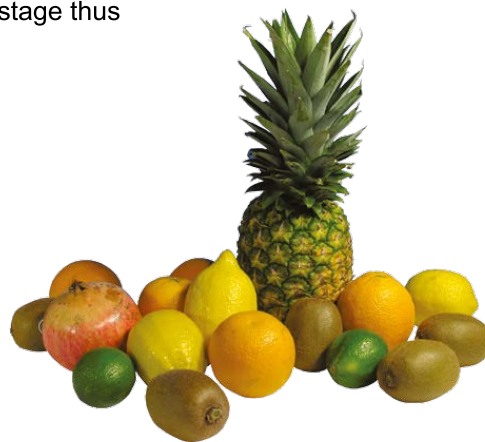

Desiccant enclosed in cap

Test strips Medi-Test Combi 10® SGL come in tubes with a desiccant containing cap. The desiccant ensures a low humidity in the tube and the longevity of the test strips. Many other packages contain a simple desiccant pack that may easily be spoiled or accidentally removed. The desiccant in the cap is . Closing the cap safely **convenient and safe** protects the test strips.

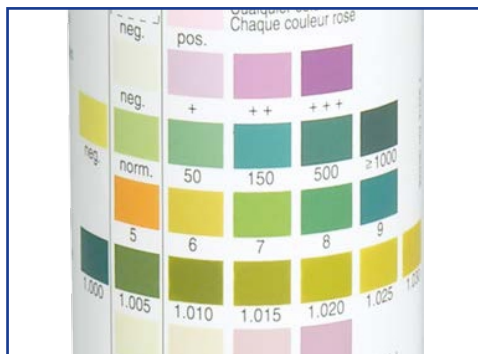

High quality color chart

The reaction colors on the test strip exactly match the colors on the label. This is ensured by a high quality printing process using specially mixed full colors. Other color charts often yield far less precise matches that may lead to inaccurate results. The evaluation of Medi-Test Combi 10® SGL is always **clear and reliable** providing perfectly accurate results.

Manufactured By:

Exclusively Marketed In India By:



MACHEREY-NAGEL GmbH & Co. KG
Neumann-Neander-Str. 6-8 · 52355 Düren · Germany
E-mail: info@mn-net.com
www.mn-net.com



Fugen Biomed
11, Dharmabhakti Complex,
Khushalwadi, Jamalpore,
Navsari-396445, Gujarat

+91 909 909 5565
contact@fugenbiomed.com
www.fugenbiomed.com